


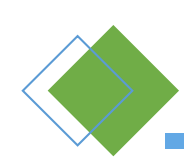

# Ucomfor Face Mask Introduction




## Face Mask

Ucomfor Face Mask is a disposable Medical grade Face Mask, main components include mask body, nose bridge, ear-loop type.

### 3 Layer Nonwoven:

- Inner Layer: Polypropylene Non-woven;
- The Middle Layer: Meltblown Non-woven; BFE $\geq$ 95% or BFE $\geq$ 98%;
- Outer Layer: Non-woven.

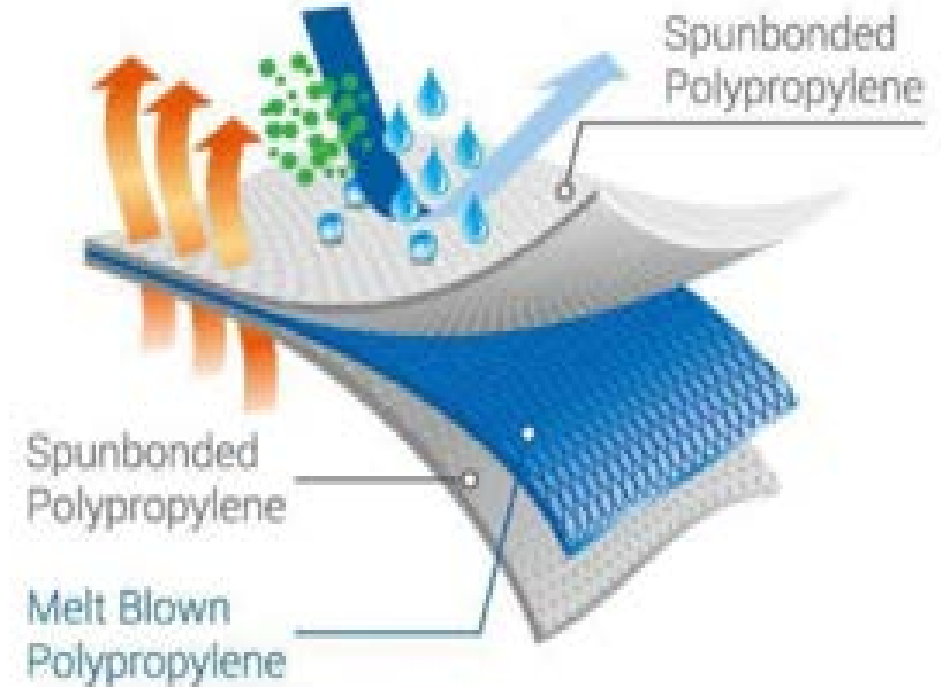



## Ucomfor® Medical Face Mask Instruction for use

### Intended use

- Type II Medical Face Mask is intended to be used to cover the mouth and nose to provide a barrier to minimize the direct transmission of infective agents between staff and patient. Type II Medical Face Mask also can be used for patients and other persons to reduce the risk of spread of infections particularly in epidemic or pandemic situations.
- Type I Medical Face Mask is intended to be used for patients and other persons to reduce the risk of spread of infections particularly in epidemic or pandemic situations.

### General description

- Complies with **EN14683 Type I or Type II** (See table).
- Meltblown Filter.
- Three Layers.
- Integrated Nose Bridge and Earloops.
- For single use.
- Do not store in direct sunlight, at extreme temperature or in high humidity.
- Keep away from getting wet.

### Direction for use

1. Place mask over nose and mouth with integrated nose bridge on top of nose.
2. Place an earloop around each ear.
3. Pull bottom of mask under chin and mold the nose bridge around the nose. The mask should cover face from the nose to underneath the chin.
4. Do not re-use single-use masks.
5. Discard single-use masks after each use and dispose of them immediately upon removal.
6. Replace mask with a new clean, dry mask as soon as they become damp/humid and dispose of properly if damaged or contaminated with blood or body fluids.

### Warnings

- This mask is not a respirator. Wearing it will NOT protect you from contracting a contagious disease, such as colds or flu.
- Type I medical face masks are not intended for use by healthcare professionals in an operating room or in other medical settings with similar requirements.

### Disposal

1. Remove the mask by using the appropriate technique (i.e. do not touch the front but remove the lace or earloop from behind).
2. After removal or whenever you inadvertently touch a used mask, clean hands by using an alcohol-based hand rub or soap and water if visibly soiled.
3. Remove mask by only handling at the earloops, then discard in appropriate waste.



Ucomfor Scientific Co., Ltd.  
www.ucomfor.com  
Lekun Road, LeYu Town, 215621  
Zhangjiagang City, Jiangsu Province,  
PEOPLE'S REPUBLIC OF CHINA

Shanghai International Holding Corp. GmbH (Europe)  
Address: Eiffelstraße 80, 20537 Hamburg GERMANY  
© Copyright 2020, Ucomfor Scientific Co., Ltd.  
All rights reserved.  
Made in China

Rev.1/2020-03-22



Storage Condition and Methods : Store in room temperature and avoid humidity

Experation Date : Two years under the specified conditions

Standard: EN14683-Type Ia & II, ASTM F2100 Level 1 & 2



*Healthy life  
begins here*

